

Instructions for use of medical scrapers from Scholz Medical GmbH

Manufacturer

Scholz Medical GmbH
Robert-Bosch-Strasse 1
83052 Bruckmühl
Germany
<https://www.scholz-instruments.com/>
info@scholz-medical.de
+49 8062 8070801



This instruction manual applies to the following products

Description	Product no.	Basic UDI-DI	Category	Rule
Sharp spoon according to Volkmann 17mm, Fig. 1	SIFK6344	426052621SIFK013K	Ila	7
Sharp spoon according to Volkmann 17mm, Fig. 2	SIFK6355	426052621SIFK013K	Ila	7
Sharp spoon according to Volkmann 17mm, Fig. 3	SIFK6366	426052621SIFK013K	Ila	7
Sharp spoon according to Volkmann 17mm, Fig. 4	SIFK6377	426052621SIFK013K	Ila	7
Curettage according to Fox length 14.5 cm	SIFK6511	426052621SIFK013K	Ila	7
Sharp double spoon according to Volkmann, 160 mm	TAFK8100	426052621SIFK013K	Ila	7
Sharp spoon according to Jansen, double spoon, 160mm	TAFK8033	426052621SIFK013K	Ila	7
Sharp spoon according to Volkmann 170mm, Fig. 0	SIFK6333	426052621SIFK013K	Ila	7
Sharp spoon according to Volkmann 170mm, Fig. 00	SIFK6322	426052621SIFK013K	Ila	7
Sharp spoon according to Volkmann 170mm, Fig. 0000	SIFK6300	426052621SIFK013K	Ila	7
Sharp spoon, double spoon, 210mm	TAFK8500	426052621SIFK013K	Ila	7

Warnings/Precautions

If you are sensitive to ethylene oxide or stainless steel, consult the medical staff before using the product.

The product must not be used on the central circulatory system and the central nervous system.

The scrapers can only be used for their intended purpose in medical disciplines by trained, medically qualified staff.

Improper storage or handling that deviates from the intended purpose may result in injury to the patient.

Store at **room temperature** and in a place protected from moisture, heat and direct sunlight, otherwise the sterility of the product cannot be guaranteed.



Always keep the scrapers out of the reach of children and other unauthorised persons.

The specified usage times must always be observed. Expired products must be disposed of.

Excessive mechanical forces from the user may cause damage to the product, which may result in injury to the patient.

Moving parts and their unintentional movement can cause injury to the user.

Reusing the scrapers can lead to bio-contamination and an increase in the bacterial load. The same applies if the instruments are used despite the packaging being damaged.



This is a scraping tool; if handled improperly or if the sharp ends are touched, the user may cut themselves or otherwise injure themselves.

Improper disposal of instruments leads to environmental pollution. Please refer to the section on proper disposal.

Intended use

In medicine, scrapers are used to remove/scrape ("curettage") tissue or wound residue. Another area of application is deep fistulas, which are scraped out with the instrument. The scrapers differ in:

- Shape of the handle
- Length of the instrument and the working area
- Shape of the spoon-like working end (round, oval)
- Grinding (blunt or sharp)

Indication: Use in routine surgical procedures in hospitals or medical practices.

Contraindication: The use is contraindicated in cases of general inoperability, previously mentioned intolerances or applications to the central circulatory and nervous systems.

Target patient group: All patient groups can be treated with sterile scrapers

Intended users: The scrapers are intended for use only by trained, medically qualified staff.

Features

There is a sterile barrier for the scrapers.

Correct application

The scrapers can only be used for their intended purpose in medical disciplines by trained, medically qualified staff. The scrapers are only used for a short time, i.e. for a period of up to several hours.

Sterility

If the sterile packaging is damaged, the scraper must no longer be used. The sterile packaging can only be opened immediately before use. If the sterile packaging is accidentally opened or damaged, please use another product with intact sterile packaging.
The product is sterilised using the ethylene oxide process.



Reuse of the product

The scrapers are designed for single use. Re-sterilisation is prohibited because there is no validated procedure for this process.

Disposal

When disposing of potentially contaminated products, regulations for preventing infection and dealing with microbiological hazards must be observed. This particularly applies to scrapers that have come into contact with tissue or body fluids.
For disposal, country-specific legislation and regulations must be observed. Our instruments are licensed for disposal via the Scholz Recycling System (SReS®).

Date of issue of the instructions for use 20/05/2025

Version: 3.0

Notice: All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is located