clevestrips PLUS


EN - Instruction for use

Product:
clevestrips – Skin Closure Strips
clevestrips PLUS – Skin Closure Strips Reinforced
 Product size: length: 30mm-200mm; width: 2mm-50mm;
Intended use:
 Product intended use: Skin Closure Strips are intended for closure fixation and nursing of wound surface after suturing, superficial layer of skin surface and the wound after removing the suture.

Product composition:
 Clevestrips - Skin Closure Strips (No- Reinforced) is composed of nonwoven and release paper.
 Clevestrips PLUS - Skin Closure Strips (Reinforced) are composed of nonwoven, polyester strength line and release paper.

Directions for use:

1. Application method

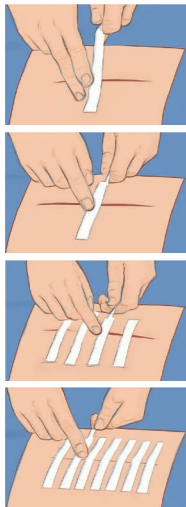A. Dry the wound, and gently stick the tape to the middle of the wound.

B. Gently press the two ends of the tape with fingers to make the application more secure.

C. Stick tapes to the wound from the middle to the sides.

D. Stick tapes to all wound areas.

2. Removal method

Loosen one end of the tape while pressing the middle with the other hand, gently and slowly remove the nonwoven dressing from the edge along the hair growing direction to avoid causing pain or discomfort to patients.

Contraindications:

Not known for contraindication

Caution:
 Product after use shall be disposed in compliance with local regulations. Re-use causes infection to the patient. Re-use can make the product viscosity drop.

Notes:
 This sterile product is for single use. No reuse is allowed.
 The safe use period should be no longer than 7 days.
 Remove the closure strip and handle it with necessary actions if infection to application area is found.
 Change for a new dressing if large amount of fluid exudes.
 Check the package before use. Do not use if there is damage, moist or expiry.

Sterilization method:
 Sterilized using ethylene oxide.

Shelf life:
 The product shelf life is 5 years. The product lot number and shelf life see the labels of the package.

Storage:
 This product should be stored seal, Keep Dry, Keep Away From Sunlight. Products should be stored in dry conditions 0-25°C.

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